

New Disinfectant Product: Oxivir ® TB

Key Points

- **Oxivir ® TB has a 28 day expiration**
- Containers must remain closed when not in use.
 - **If the lid remains open, the wipes expire in 24 hours**
- **Do not discard the containers**
 - The containers are intended for reuse

Usage Instructions

- Check expiration on container lid before use
- Keep container closed when not in use
- The dwell time for disinfection is one (1) minute
- For **Enteric Precautions** the dwell time is three (3) minutes
- Used wipes are disposed in the garbage waste receptacle

Supply Workflow

- If the Oxivir ® TB expiration date has passed, remove the container from the clinical environment
 - ❖ Do not discard the Oxivir ® TB containers
 - ❖ Place the container in the soiled utility area or other designated area
 - ❖ Materiel Services will collect the used containers for cleaning performed by SPD
- Replace the expired container with a new container from supplies stored in clinical area drop shipment, clean utility room or other designated area

